430-086  
C/132394  
August 2017

# **STERNAL CABLE SYSTEM INSTRUMENTS**

# DIRECTORY

<b>en</b>	..... Page 1
<b>cn</b>	..... 第 4 页
<b>da</b>	..... Side 6
<b>nl</b>	..... Pagina 9
<b>fi</b>	..... Sivu 12
<b>fr</b>	..... Page 15
<b>de</b>	..... Seite 18
<b>el</b>	..... σελίδα 21
<b>it</b>	..... Pagina 24
<b>no</b>	..... Side 27
<b>pt</b>	..... Página 30
<b>es</b>	..... Página 33
<b>sv</b>	..... Sida 36
<b>tr</b>	..... Sayfa 39

## IMPORTANT PACKAGE INFORMATION

### INTENDED USE

Sternal surgical instruments are for use only with Pioneer's Sternal Cable System implants.

### IMPORTANT

This information is intended to aid in using the Pioneer Sternal Cable System and is not a reference for surgical technique. Refer to the surgical technique manual for instructions for implantation. Only the instruments designed for the Pioneer Sternal Cable System should be used during the procedure.

### INSTRUMENT CARE AND CLEANING

1. Instruments are provided nonsterile.
2. Prior to initial and each use, sterilize surgical instruments according to AORN recommended practices.
3. As soon as possible after each use soak the instruments in cool, distilled water.
4. Instruments should be hand scrubbed using a nearly neutral pH, low sudsing, blood-dissolving detergent. The detergent should dissolve in cool water and must be completely rinsable. Scrubbing should be done under water to prevent aerosol formation.
5. Lubricating the instruments is recommended.

### WARNINGS AND PRECAUTIONS

1. Avoid over tensioning the cable construct.
2. Saline is corrosive to stainless steel and should not be used.
3. High pH level detergents may cause pitting and tarnishing of the instruments
4. Instruments are subject to damage during use as well as long-term potentially damaging effects such as wear. Damage may result in significant risks to safety and/or inability to function as intended.
5. If instruments are damaged or broken during use, metal fragments can be viewed by radiographic assessment. It is the surgeon's responsibility to carefully consider the risks and benefits of retrieving the fragments.
6. If the fragment is retained in the patient, it is recommended that the surgeon advise the patient of specific information regarding the fragment material, including size and location and the potential risks associated with the retained fragment.

### INSTRUMENT STERILIZATION

<u>Method</u>	<u>Cycle</u>	<u>Temperature</u>	<u>Exposure Time</u>	<u>Dry Time</u>
Steam	Pre-vacuum (Wrapped)	132° C (270° F)	6 Minutes	20 Minutes

Use of an FDA cleared wrap is recommended to ensure product sterility.

These parameters are validated to sterilize only this device. If other products are added to the sterilizer, the recommended parameters are not valid and new cycle parameters must be established by the user. The autoclave must be properly installed, maintained, and calibrated. Ongoing testing must be performed to confirm inactivation of all forms of viable microorganisms.

## Sternal Crimper/Tensioners

### Instrument Cleaning and Maintenance Reprocessing Instructions

This reprocessing procedure is valid only for the Pioneer Sternal Crimper/Tensioner.

- Prior to and during use, including reprocessing, inspect instruments for:
  - Damage such as, but not limited to, wear, discoloration, corrosion, cracking, fracture, or unrecognizable markings.
  - Proper function including, but not limited to, sharpness, movement of hinges and couplings, joint stability, and legible markings.
  - Detailed instructions are available per the surgical technique.
- Instruments that show signs of damage or an inability to function should not be used and should be returned to the manufacturer.

#### Procedure

- 1) At Point of Use: Immediately following use remove gross debris and soil from instrument using a damp clean cloth.
- 2) In transit: Immediately after gross debris is removed instrument should either be immersed in a basin of water or covered with a damp cloth so as to prevent the instrument from drying prior to full cleaning process being completed.
- 3) Presoak: Soak instrument for a minimum of 20 minutes in an enzyme solution, for example – Enzol. Mix the enzymic solution per the manufacturers instructions. Presoak will reduce proteinaceous material from the instrument.
- 4) Manual Clean: Thoroughly brush all parts of instrument with a soft bristled non-abrasive brush until all visible soil is removed from the instrument. Particular attention should be paid to lumens, crevices and other hard-to-clean areas.
- 5) Rinse: Rinse instrument with Deionized water for a minimum of three minutes. Particular attention should be paid to lumens, crevices, serrations, joints and other hard-to-clean areas.
- 6) Mechanical Clean: Prepare ultrasonic cleaner with an enzymic detergent as per the manufacturers instructions. Completely submerge instrument in sonicator and sonicate for a minimum of 10 minutes.
- 7) Rinse: Rinse instrument with Deionized water for a minimum of three minutes. Particular attention should be paid to lumens, crevices and other hard-to-clean areas.
- 8) Inspect: Visually inspect instrument for residual debris or soil. If any is present, repeat steps 4 – 7 above until there are no visible signs of debris or soil on the instrument.
- 9) Lubricate: Lubricate the instrument with a water-based lubricant that contains bacteriostatic agents. A water soluble lubricant is used so as to not impede steam penetration during sterilization.

Manual application of water –based lubricant: Immerse clean and opened instrument

into lubricant for minimum of 45 – 60 seconds. While the instrument is immersed in the lubricant move the mechanisms so as to work the lubricant into the moveable components. Do not rinse or wipe the instrument following the application of lubricant.

**LIMITED WARRANTY.** Pioneer Surgical Technology products are sold with a limited warranty to the original purchaser against defects in workmanship and materials. Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed.

### **FURTHER INFORMATION**

Recommended surgical operative techniques and uses for all instruments and cable products are available at no charge upon request.

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Pioneer Surgical Technology, Inc.
Marquette, MI 49855 U.S.A.
RSQR Ltd.