WIRE TWISTER

PRODUCT DESCRIPTION:

The Wire Twister is used to secure and tighten DoubleWire™ Sternotomy Sutures during surgical procedures. The Wire Twister is supplied non-sterile.

INDICATIONS FOR USE:

The Wire Twister is a tool which aids in the securing and tightening of metal sutures.

WARNINGS:


1. The Wire Twister is supplied non-sterile and must be sterilized prior to use.
2. Overtightening of a suture may result in unintended tissue or bone damage.
3. Do not use if any damage, cracks, or deterioration is observed.

PRECAUTIONS:

1. Under tightening of a suture may result in insufficient holding strength for the desired surgical effect.
2. Autoclave only. Dry Heat processing will degrade the twister and should not be performed.

INSTRUCTIONS FOR USE:

1. Prior to use in the sterile operating field, the twister must be sterilized in accordance with the user's institution standard operating procedures.
2. Device integrity should be checked before each use. Do not use if damaged, cracks, or deterioration.
3. Insert the hook of the Wire Twister into the loops of a suture. Using upward pressure, rotate the Wire Twister to tighten the suture. (See illustrations and detailed instructions below.)

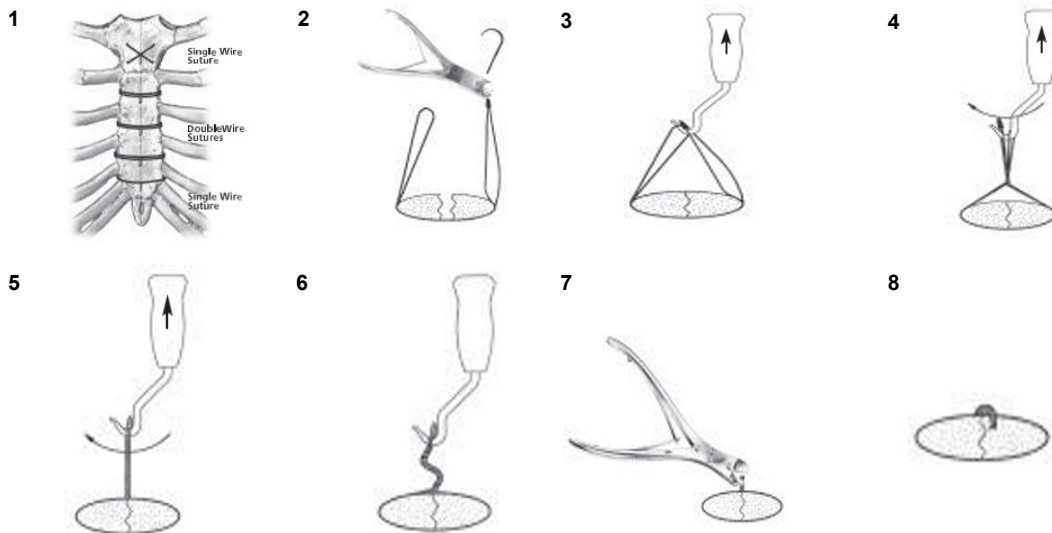1. When installing the DoubleWire™ sutures, the curved needle is passed around each side of the mesosternum so that an equal portion of the loop appears on both sides. Care should be taken to avoid the internal mammary artery. Individual Single Wire sutures are placed through the manubrium and around the xiphoid.
2. The curved needle is removed from the wire by cutting the single wire strand with wire cutters immediately above the DoubleWire™ sternotomy suture junction. After installing the chosen number of DoubleWire™ and Single Wire sutures, begin sternal closure by twisting the single wire at the manubrium and xiphoid areas.
3. For DoubleWires™, the DoubleWire™ Twister hook is passed through the two open portions of each loop on opposite sides of the sternum. Both loops must be located in the lowest portion of the twister hook prior to twisting.
4. Upward pressure is applied with the twister to draw each loop up evenly on each side of the sternum. While maintaining this upward pressure of the twister, then rotate to braid the two pair of wires together. The anterior plates of the sternum should be approximated with the index finger of the hand while twisting the wires.
5. The twisting tool is rotated until the two loops have been fully twisted into a single cable.
6. At the onset of the double cabling, optimum tension has been achieved and twisting should be discontinued. Return to the single monofilament sternum wires to complete tightening.
7. The wires are cut with double action wire cutters (i.e. Aesculap #LX156) leaving about ¼ to 1 cm.
8. Then turn down the twisted wire in the direction of the twist to bury the tip of the wire. The twisted wire is then flattened onto the anterior plates of the sternum with the blunt tip of a needleholder.

STERILIZATION:

1. The twister should be cleaned with soapy water prior to sterilization to ensure contamination is removed prior to autoclaving.
2. The Wire Twister may be steam autoclaved in accordance with generally accepted practices such as AAMI/ANSI Good Hospital Practice: Steam Sterilization. The Wire Twister can withstand a steam cycle of up to 10 minutes at 132°C (270°F).
3. The Wire Twister has been validated for up to 20 steam sterilization cycles.
4. Device integrity should be checked after each sterilization cycle for damage, cracks, or deterioration.

STORAGE:

The Wire Twister should be stored at room temperature not to exceed 50° C (120° F).

PRODUCT LABEL SYMBOLS	
	Do not use if package is damaged

The Instructions for Use and a comprehensive symbol glossary can be found at <http://www.aemedical.com/quality/>

	A&E MEDICAL CORPORATION 5206 Asbury Road, P.O. Box 758 Farmingdale, NJ 07727 USA	EC REP	CE
		BIOMED, S.A. Parque Tecnológico de Madrid C/ Einstein, 3 Tres Cantos 28760 Madrid, Spain	

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